



Sculptra®

SUBTLE.  
Gradual.  
LASTING.\*

**DISCOVER SCULPTRA.**

\* Results may last for over 2 years based on a 25-month clinical trial.

PART OF THE GALDERMA  
AESTHETICS COLLECTION

# COLLAGEN HELPS KEEP YOUR SKIN LOOKING *youthful & smooth*

## **OVER TIME, OUR BODIES MAKE LESS COLLAGEN, LEADING TO THE VISIBLE SIGNS OF AGING.**

Collagen is a key structural component under the skin that provides volume and gives shape to the face, keeping skin looking youthful. Over time, our bodies make **less collagen** and our skin loses its elasticity and volume. This, along with environmental factors like sun exposure, can lead to the appearance of wrinkles, lines and folds.

Anti-aging creams containing collagen **cannot** get below the surface of the skin to replace lost collagen. *But, Sculptra can.*

**Helps address the effects of skin aging in a subtle way.**



## **WITH AGE, THESE CHANGES IN STRUCTURE CAUSE CHANGES IN APPEARANCE.**

Collagen loss, bone remodelling, and fat redistribution all contribute to changes that result in a less youthful appearance.

Sculptra®

# The SCULPTRA *difference*

## REJUVENATION FOR THE MODERN AGE.

In contrast to other wrinkle fillers, Sculptra works gradually with your body to replace collagen lost from the aging process. As a result, volume is restored and wrinkles and folds are smoothed – creating a more youthful, natural, subtle look that can last over 2 years!\*

Sculptra is suitable for increasing the volume of depressed areas, particularly to correct skin depressions, such as in skin creases, wrinkles, folds and scars, and for skin aging.


Sculptra is also suitable for large-volume corrections of the signs of facial fat loss (lipoatrophy).

## HOW DOES SCULPTRA WORK?

- + Works gradually for a natural look
- + Restores volume
- + Smooths out wrinkles, creases and folds


## SCULPTRA WORKS BY TARGETING AN UNDERLYING CAUSE OF SKIN AGING.

### AGING SKIN

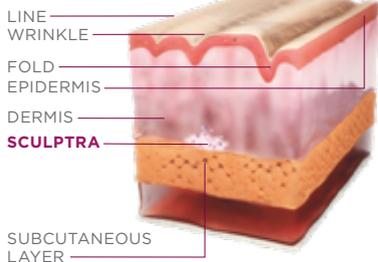

Sculptra is injected into the deep dermis (as shown in the illustration) or subcutaneous layer.

### SCULPTRA

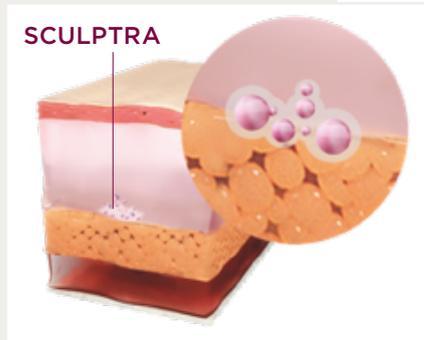

The injected volume helps fill out the wrinkles and/or correct volume defects.



Over time, the Sculptra particles are degraded, but the collagen deposits remain and provide gradual, long-lasting improvements of wrinkles and volume defects.

\* Clinical trial ended at 25 months; Narins RS, *et al.*, 2010.

Sculptra®

# *Gradual,* LONG-LASTING, NATURAL-LOOKING *results\**

Used in  
**150,000+** patients

•

**No** human,  
animal  
or bacterial  
components

•

Absorbed  
**naturally**  
by the body

•

**No** lasers  
or incisions



TEMPLE HOLLOW


CHEEK HOLLOW

CHEEK DEPRESSION

NASOLABIAL FOLD

MARIONETTE LINE

CHIN FOLD/WRINKLE

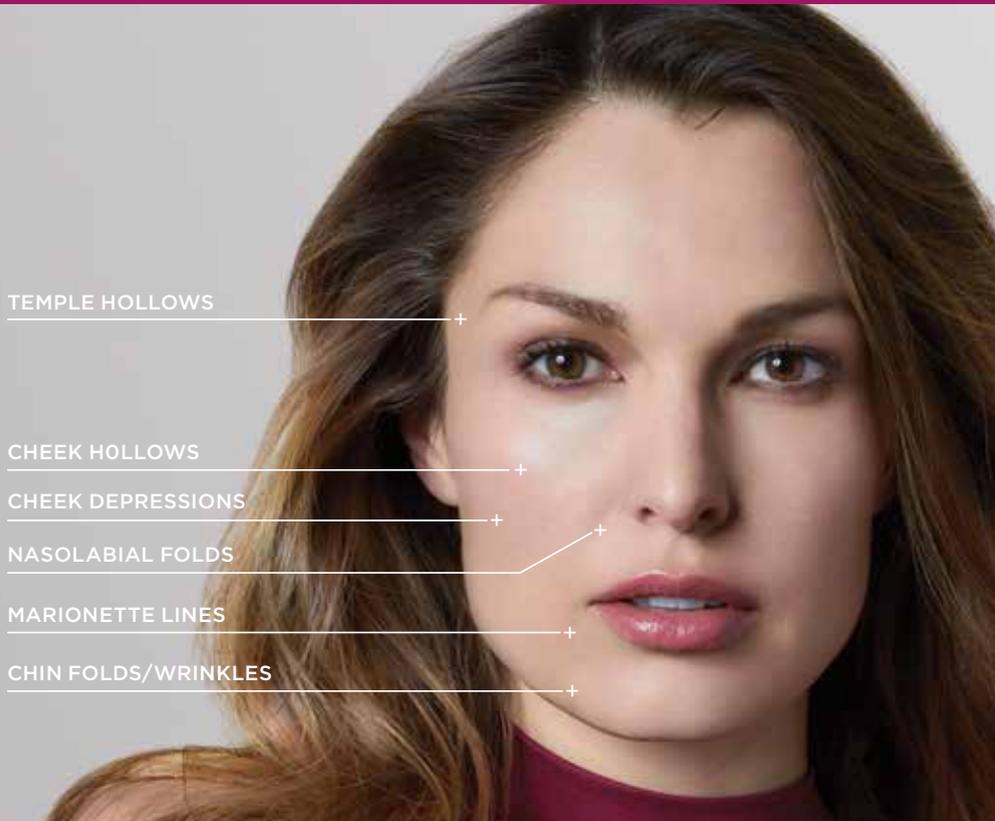

SCULPTRA MAY NOT BE FOR EVERYONE.  
TALK TO YOUR HEALTHCARE  
PRACTITIONER TO SEE IF SCULPTRA  
IS RIGHT FOR YOU.

\* Clinical trial ended at 25 months; Narins RS, *et al.*, 2010.

Sculptra®



# REAL *results*

*"I want to look better,  
younger - but still like me.  
With Sculptra, I got the  
natural, youthful look  
I wanted without feeling or  
looking like someone else."*


**ELISSA**  
5 vials of Sculptra  
over 3 sessions



Before



Session 2 results

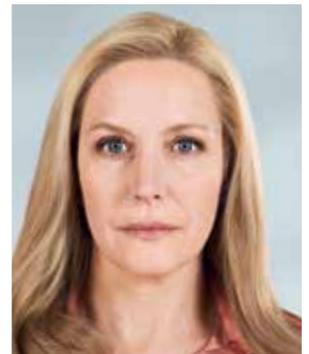

Month 7\*

\* Individual results may vary. Photos have not been retouched.

Sculptra®

# FREQUENTLY ASKED *questions*

## HOW IS SCULPTRA DIFFERENT FROM FILLERS?

Traditional fillers are liquid or gel substances, such as hyaluronic acid. They work by adding volume to smooth out wrinkles, lines and folds, with immediate results. **They do not stimulate the body to replace collagen.**

Sculptra is biodegradable and biocompatible, and works deep under the skin to gradually **replace lost collagen** to correct skin creases, wrinkles, folds and scars, and for skin aging. Results emerge over a few weeks, for a more subtle, natural and youthful appearance that can last over 2 years.<sup>1\*</sup>

## HOW MANY TREATMENTS ARE REQUIRED?

The number of treatments you need depends on various factors, including the surface area to be treated. At your initial treatment, your Sculptra physician will discuss with you how many follow-up treatments may be required.

## HOW LONG DOES IT TAKE TO SEE RESULTS?

Because Sculptra works over time to replace collagen – the underlying cause of facial aging – results are not immediate. You will see a **gradual change** in the appearance of your skin over several weeks as the treatment effect occurs, creating a more youthful look.

## HOW LONG DOES SCULPTRA LAST?

Sculptra treatment results differ from person to person. Results can last over **2 years**.<sup>1\*</sup> Touch-up treatments may be needed to maintain the desired effect. Your healthcare practitioner will discuss the right treatment plan for you.

\* Clinical trial ended at 25 months; Narins RS, *et al.*, 2010.

## IMPORTANT SAFETY INFORMATION

- + Sculptra should not be used in any person who has hypersensitivity to any of the components of the product.
- + Use of Sculptra in any person with active skin inflammation or infection in or near the treatment area should be deferred until the inflammatory or infectious process has been controlled.
- + There are several precautions that are important to discuss with your healthcare professional prior to treatment. Inform your healthcare professional if you have any of the following:
  - If you are using anticoagulants as you may run the risk of a hematoma or localized bleeding at the injection site.
  - If you have a known history of or susceptibility to keloid formation or hypertrophic scarring.
  - If a laser treatment, chemical peel or any other procedure based on active dermal response is performed before or after the treatment. There is a possible risk of eliciting an inflammatory reaction at the implant site.
- + Injection procedure reactions to Sculptra have been observed consisting mainly of hematoma, bruising, edema, discomfort, inflammation, erythema, transient bleeding from an area the size of the point of the needle or transient pain, localized redness at the injection site and ecchymosis. The most common device-related adverse effect was the delayed occurrence of subcutaneous papules, which were confined to the injection site and were typically palpable, asymptomatic and non-visible.
- + Rare but serious adverse events associated with the intravascular injection of soft tissue fillers in the face have been reported and include temporary or permanent vision impairment, blindness, cerebral ischemia or cerebral hemorrhage, leading to stroke, skin necrosis, and damage to underlying facial structures. You should discuss the potential treatment risks with your healthcare professional so you are aware of signs and symptoms of potential complications.
- + After the treatment, minimize the exposure of the treated area to excessive sun and avoid UV lamp exposure until any initial swelling and redness have been resolved.
- + This product should only be administered by trained healthcare professionals.
- + This product has not been tested in pregnant or breastfeeding women.
- + Not for use in children under 18 years of age.
- + Injection-related adverse events such as bruising, erythema, itching, swelling, pain and tenderness may occur. These side effects generally resolve spontaneously within one week after injection. Other potential side effects can occur with dermal filler injections. You should discuss the potential treatment risks with your healthcare professional.
- + **Discuss all potential risks with your healthcare professional prior to treatment. This is only a partial list of safety information. For complete product and safety information, visit [Sculptra.ca](http://Sculptra.ca).**

Sculptra®

look YOUR BEST,  
feel YOUR BEST

## FOCUSED ON SKIN.

**Galderma is the world's largest independent global skin health company**

A leading innovator in aesthetics with over 23 years of experience, Galderma is a trusted partner for healthcare practitioners and patients around the world.



To learn more about Galderma, please visit [Galderma.ca](http://Galderma.ca)

**Want gradual, natural-looking results that last?\***  
**Ask your healthcare practitioner about Sculptra today.**

Learn more at [Sculptra.ca](http://Sculptra.ca)

# Sculptra®

**REFERENCE:**

1. Narins RS, *et al.* *J Am Acad Dermatol* 2010;62:448-462.

\* Results have been shown to last over 2 years. Clinical trial ended at 25 months. Narins RS, *et al.*, 2010.

 **GALDERMA**

©2020 Galderma Canada Inc. All rights reserved.  
All trademarks are the property of their respective owners.  
SCU-007-0120E